


IMPORTANT INFORMATION

Dispensing Liquid Recommendations Compatible for use with miscea Systems.

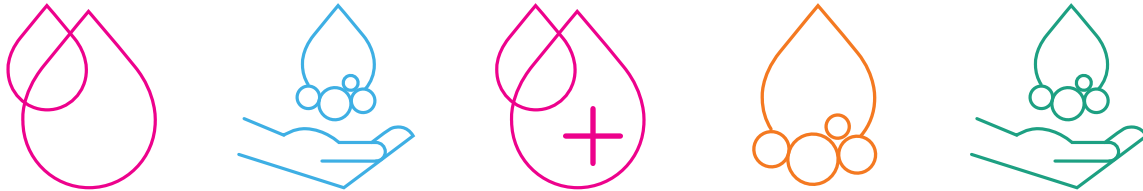

We recommend using miscea refill products with all miscea systems. The liquids provided by miscea have undergone numerous compatibility tests and were selected based on a number of different criteria to meet the general needs of users within the medical field.

As many organisations may have specific requirements regarding the type of soaps and disinfectants used within their facility, we understand there is a need to use miscea systems with liquid soaps and disinfectants provided by other manufacturers. In these instances and similar, it is possible to use miscea systems with commercially available liquid soaps, disinfectants and lotions.

In order to ensure the proper function and prolong the life of the system, please take note of the following points below.

1. Which types of liquids can be used with miscea systems?

Commercially available soaps, disinfectants and lotions within a certain viscosity range can be safely used with miscea systems.

2. What is the allowable viscosity range of liquid soap, disinfectant and lotions?

The viscosity of the soap, disinfectant or lotion used with miscea systems can be between 5 - 2500 mPas. To give an example of the various viscosity ranges of different liquid products, please see the table below.

Water: 1 mPas

Milk: 3 mPas

Cooking oil: 1,000 mPas

Honey: 10,000 mPas

3. What is the allowable pH value for the selected liquid soap, disinfectant or lotion?

The pH level of the liquid soap, disinfectant or lotion should be between 5 and 9.

4. What other points should be considered when selecting a compatible soap, disinfectant or lotion for the miscea system?

The selected soap, disinfectant or lotion should also not contain any abrasive materials or particles in the solution. Having abrasives in the liquid will deteriorate the tubes in the system and reduce the life of the system.

For maintenance purposes, we recommend using the miscea TubeCleanse solution to clean and disinfect the dispensing tubes. This can be purchased through miscea or your miscea representative.

For further information, please contact miscea GmbH by telephone on **+49 33701 3553 0** or by email at **info@miscea.com**.